




Olympus EndoDet Pro, EndoDis Pro and EndoAct Pro

EndoPro PAA Process Chemicals

Olympus EndoDet Pro, Olympus EndoDis Pro and Olympus EndoAct Pro combine powerful performance with good endoscope compatibility and simple handling.

The Olympus EndoPro PAA process chemicals are approved for flexible endoscopes of all considerable brands and support a long lifetime of endoscopes and ETD machines.

	Olympus EndoDet Pro	Olympus EndoDis Pro	Olympus EndoAct Pro
Product Description			
	Olympus EndoDet Pro is a slightly alkaline detergent based on a special enzyme-surfactant combination offering excellent cleaning performance.	Olympus EndoDis Pro based on PAA and Olympus EndoAct Pro are highly concentrated and ensure in their combined application uncompromising antimicrobial efficacy under material-compatible application conditions.	
Composition	Non ionic and anionic surfactants, alkaline agent, protease, preservative, excipients	Active ingredient: Peroxyacetic acid Other ingredients: Hydrogen peroxide, acetic acid, chelating agent	Phosphates, potassium hydroxide, chelating agent
Spectrum of Efficacy	Olympus EndoDet Pro effectively reduces organic debris and protein residues.	<p>The efficacy of disinfection with Olympus EndoDis Pro in combination with Olympus EndoAct Pro was tested in accordance with the norms listed below in suspension tests and practical tests considering the application parameters of the ETD EndoPro PAA processes:</p> <p>Bactericidal activity: European Norm (EN) 13727 and EN 14561 Yeastcidal activity: EN 13624 and EN 14562 Tuberculocidal activity: EN 13624 and EN 14562 Mycobactericidal activity: EN 14348 and EN 14563 Sporocidal activity: EN 17126 Virucidal activity: EN 14476, EN 17111 and according to the guidelines of the German Association for the Control of Virus Diseases (DVV) and the Robert Koch Institute (RKI)</p>	
Directions for Use	Olympus EndoDet Pro, Olympus EndoDis Pro and Olympus EndoAct Pro are all supplied as a concentrate. Place the containers into your compatible ETD washer-disinfector according to the instructions for use. The concentrates will then be automatically dosed into the machine process by the dosage pumps.		
Application Parameters	Effective concentration: 0.6% (6 ml/l) Contact time: 3 min Temperature: 35°C	Effective concentration: 1.2% (12 ml/l) Contact time: 5 min Temperature: 39°C	Effective concentration: 0.6% (6 ml/l) Contact time: 5 min Temperature: 39°C

Olympus EndoDet Pro, EndoDis Pro and EndoAct Pro

	Olympus EndoDet Pro	Olympus EndoDis Pro	Olympus EndoAct Pro
Application Parameters	Application parameters may vary according to ETD process settings.		
Presentation	4.5 l canister (sales unit: 3 canisters)	4.5 l canister (sales unit: 3 canisters)	4.5 l canister (sales unit: 3 canisters)
Shelf Life*	24 months	18 months	24 months
Biocompatibility	Biocompatibility of process chemical residues has been evaluated according to ISO 10993. Simulated use tests showed no cytotoxic potential of process chemical residues.		

*For storage information, refer to the relevant Material Safety Data Sheet (MSDS). The expiry date is shown on the label on the container. The date is stamped behind the hourglass symbol.

Note: Protective equipment and clothing should always be worn when handling chemicals.
Refer to the relevant Material Safety Data Sheets (MSDS).

As medical knowledge is constantly growing, technical modifications or changes of the product design, product specifications, accessories and service offerings may be required.

DISTRIBUTOR

OLYMPUS

OLYMPUS EUROPA SE & CO. KG
Postbox 10 49 08, 20034 Hamburg, Germany
Wendenstrasse 20, 20097 Hamburg, Germany
Phone: +49 40 23773-0, Fax: +49 40 233765
www.olympus-europa.com

MANUFACTURED BY

Ecolab Deutschland GmbH, Ecolab-Allee 1
40789 Monheim am Rhein, Germany
Phone: +49 (0)2173 599 1000, Emergency number: +32 (0)3 575 5555